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KA1001

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
KA1001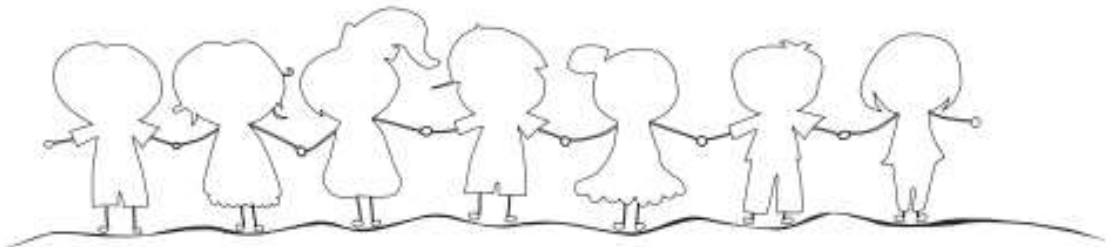

Safety instructions for use

- This product is only used to clean the nasal cavity and derivatives of the baby's nasal cavity.
- Prevent solid and hard objects above 3mm from entering the nozzle.
- Please don't throw away randomly, so as not to damage the product.
- Please use the accessories provided by the manufacturer to avoid damage to the machine due to inconsistent accessories.
- This product is not a toy, please keep it in a safe place and do not let children play.

Precautions before use

- It is recommended to sterilize the suction nozzle and collection cup before use, such as soaking in boiling water at 212°F / 100°C for a few minutes and then install it on the main body of the equipment for use. Do not use high-temperature machines such as microwave ovens for detoxification..
- Do not immerse the whole product in liquid.
- Please check whether the seal is damaged and the accessories are complete.
- Wash your hands before use


Features

- This product is a brand-new nasal cavity cleaning product, which is soft, efficient and easy to clean. It can clean the nasal cavity of babies and children under safe and comfortable conditions.
- The product is equipped with three levels of suction strength, which can be adjusted according to the nasal cavity.
- The product can emit beautiful music, allowing babies to clean their nasal cavity in a relaxed atmosphere.
- The blister cup and suction nozzle made of food-grade materials are safe and hygienic, easy to operate, and easy to keep the baby's nasal cavity clean


Product preservation method

- Please use cleaning agents such as alcohol or detergent to clean this product.
- The suction nozzle and collection cup can be disinfected with boiling water or steam, but cannot be disinfected with a microwave oven.
- The main body of the product is not waterproof. Do not wash with water or clean at high temperatures. Please use a soft wet towel to wipe the main body of the product for cleaning.
- Avoid storing the product in direct sunlight and humid environment.
- Avoid using corrosive chemicals to clean this product

Product structure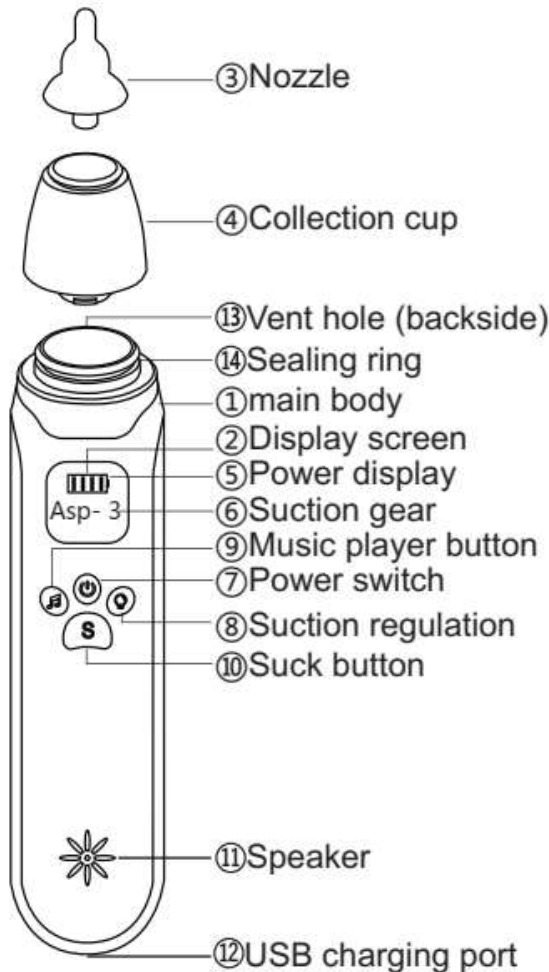


▲ Three replaceable nozzles

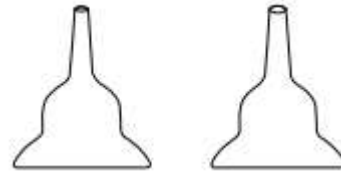Round hole nozzle

Special for internal nasal cavity
There are two sizes, please use the appropriate nozzle according to the child's nostril size.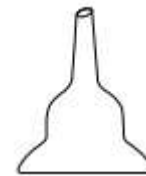

Oblique mouth nozzle

Used to clean snot or suck out thick secretions.

- Operation instructions

9. Turn on/off: click ⑦ on/key to turn on the device, and click again to turn off the device to cycle;
10. Gear position adjustment: Click the ⑧ gear position adjustment button to set the gear position intensity, 1-3 gears are cycled in turn;
11. Music playback: Click the ⑨ music playback button to turn on the music playback, and click again to turn off the music;
12. Inhale button: click the ⑩ inhale button to turn on the inhalation function to clean the nasal cavity, and click again to turn off the inhalation function;
13. Device charging: Use DC 5V 1A to connect to the USB charging socket to charge the device.

Note (please read carefully before use)

- ▲ During use, make sure that the baby's mouth is opened to ensure air circulation in the nasal cavity to avoid discomfort caused by negative pressure on the head.
- ▲ When the nasal aspirator is working, do not block the other nostril, so as to avoid negative pressure on the head and cause discomfort.
- ▲ Please pay attention to clean this product before each use.
- ▲ Disinfect the suction nozzle and transparent collection cup after use to prevent cross-infection.
- ▲ Do not immerse the entire product in liquid.
- ▲ When the capacity of the collecting cup reaches half in use, it should be cleaned up in time before continuing to use it to prevent nasal dirt from being sucked into the motor and damage the product.

Product parameter

Product Name: Baby Nasal Aspirator

Product Model: KA1001

Working voltage: 3.7V

Charging voltage: DC 5V 1A

Maximum power: 1.5W

Battery capacity: 500mAH

Product net weight: 150g

Product size: L207×W41.5×H49mm

Trouble shooting

If the following problems occur, please follow the "Troubleshooting Guide". If you cannot solve them, please contact our after-sales service


Problem	Solution
No suction	1.Check whether the nozzle is installed in place 2.Check whether the collection cup is in place 3.Check if the vent is blocked 4.Check whether the sealing ring is intact
Device can't start	Please charge before use

Baby nasal aspirator


Quick guide

Component description

- Main body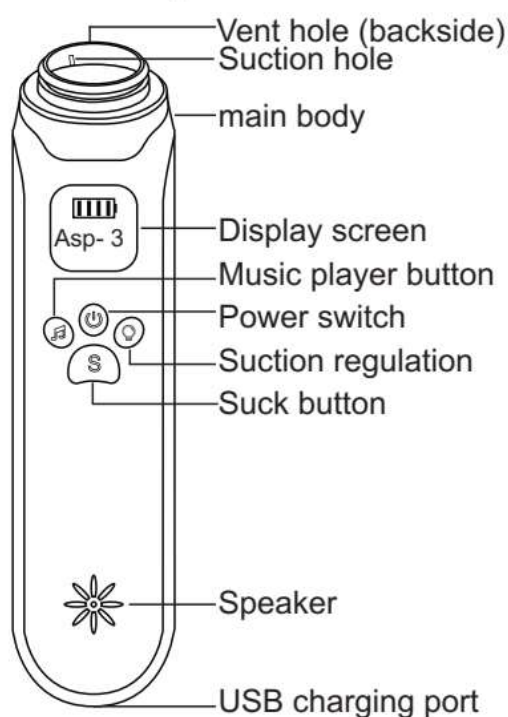


- Three replaceable nozzles

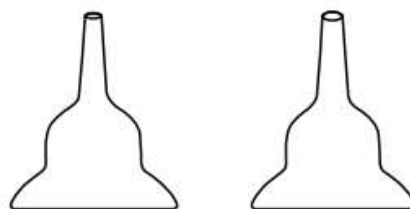Round hole nozzle

Special for internal nasal cavity


There are two sizes, please use the appropriate nozzle according to the child's nostril size.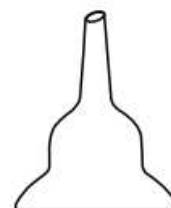

Oblique mouth nozzle

Used to clean snot or suck out thick secretions.

- Accessories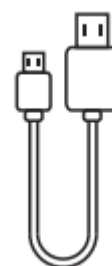

USB charging cable


Steps


14. Buckle the base of collection cup into the main body of the device, turn it clockwise to tighten, and then install the sealing cover on the base.


15. Snap the protective cover of the collection cup into the main body and turn it clockwise to tighten it.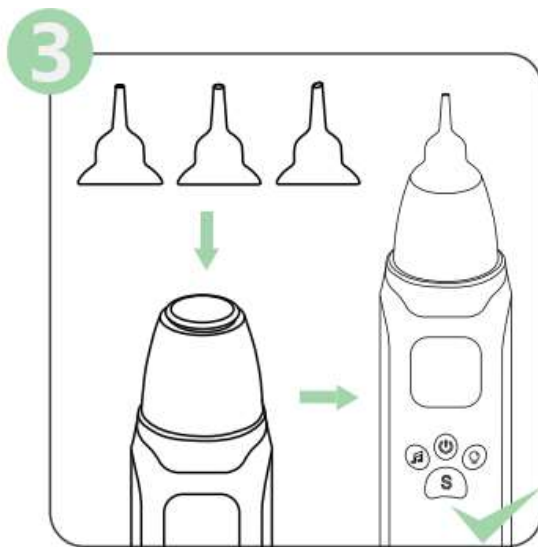

16. Select desired nozzle and install it on the collection cup.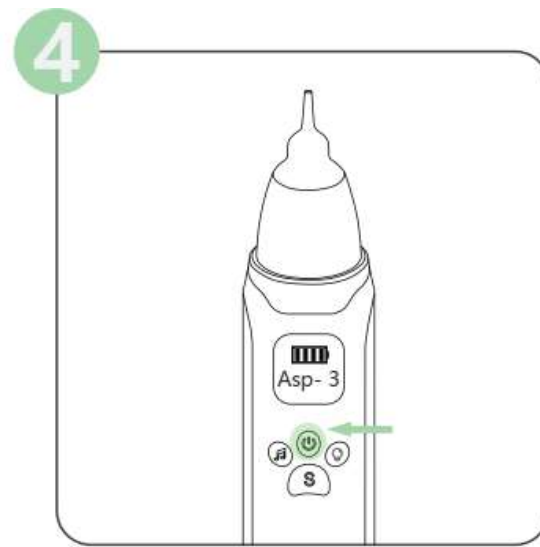

17. Press the power button to start the device.

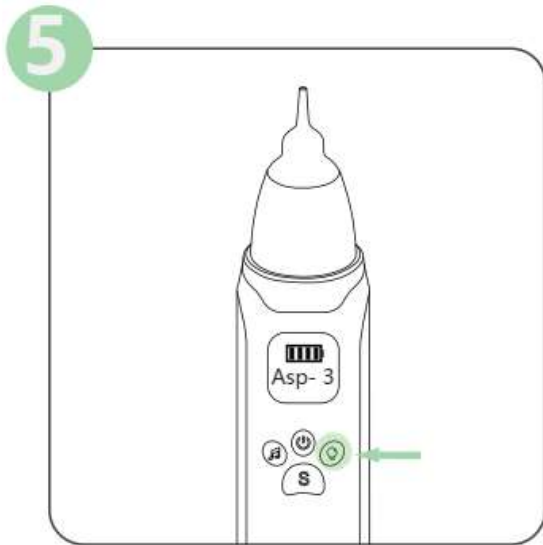18. Adjust the gear to the desired intensity.

19. Press the music play button to play music. This function is optional. Beautiful music can attract a baby's attention and make the process of sniffing easier.


20. Press the suck button to turn on the inhale function.

21. Place the suction nozzle in the nasal cavity to clean the nasal cavity.

Use details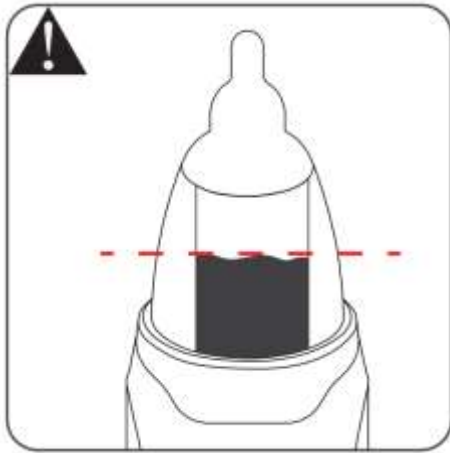

Note: When cleaning the nasal cavity, when the effusion in the collection cup exceeds half of the volume, please remove the collection cup in time and clean it before continuing to use it, otherwise it will easily lead to backflow, and it will be difficult to clean after the backflow.

Tip: Please use boiling water or steam to disinfect the suction nozzles and collection cup before use to avoid bacterial infection. Note that the microwave oven cannot be used for disinfection, available to put detachable parts in the dishwasher, water temperature cannot exceed 212°.

Note: The main body of the product is not waterproof. Do not rinse with water or clean at high temperature. Please use a soft damp towel to clean it.

Tip: After use, please clean the suction nozzle and collection cup with hot water in time to prevent bacteria from breeding.

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